

# MANUAL

## **BIX/FL ADVANCED GYNECOLOGICAL EXAMINATION TRAINING SIMULATOR**



**Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.**

**Ada Med Supply Limited**

## BIX/FL ADVANCED GYNECOLOGICAL EXAMINATION TRAINING SIMULATOR

### **The purpose of the model:**

The Advanced Gynecological Examination training Simulator is according to the teaching requirement of department of gynaecology, the design of it lay emphasis on simulated effect. For the demonstration of the teacher and the students training in diagnosis and vaginal speculum examination. Applicable to the operation teaching and practice of medical colleges, maternal and child health hospital.

### **The main function:**

The model is the adult female torso lower abdominal, pelvic by two parts. Can make the following functions:

- Normal uterine palpation, visual examination (not production, multiparity, pregnant, the elderly)
- Bimanual gynecological, three bimanual examination.
- Vaginal speculum and colposcopy examination.
- Placement and removal of IUD.
- Uterine pregnancy and postpartum uterine examination.
- Observation of uterine, ovarian, fallopian tube, round ligament and other anatomy in pelvic.
- Can adjust the tilt angle position of the uterus.
- Pelvic measurement.

### **Method of use:**

1, Vaginal speculum examination: vaginal speculum insert vaginal, vaginal dilation, exposure of cervix.

2, The double rash, three rash, anal referral examination: can display the vaginal length, the shape of cervix, uterus and appendages.

**Product maintenance:**

- 1, Keep clean, after the end of the training operation, the model should be to scrub clean with a clear damp cloth, prohibited scrub use chemical solvents.
- 2, Please clean up the model when long time not use it, container stored in ventilated, dry and cool place, to extend the service life.
- 3, the model within one year from the date of purchase or in the warranty period prescribed in the contract, If there is any fault and other quality problems because any manufacturing process or the original (except for man-made outer), the company is responsible for free repair service. And a commitment to lifelong maintenance (cost the users themselves).

**Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.**

**Ada Med Supply Limited**

---

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd. Building 3,  
No.1288 kungang Road, Xiaokunshan Town, Songjiang District, Shanghai, China

Tel: +86-13383897707

Fax: 021—57722933

Post code: 201612

E-mail: [edith@adahealthy.com](mailto:edith@adahealthy.com)

Website: [www.adahealthy.com](http://www.adahealthy.com)