


# MANUAL

## **BIX - H70 ADVANCED BASIC NURSING TRAINING MODEL**

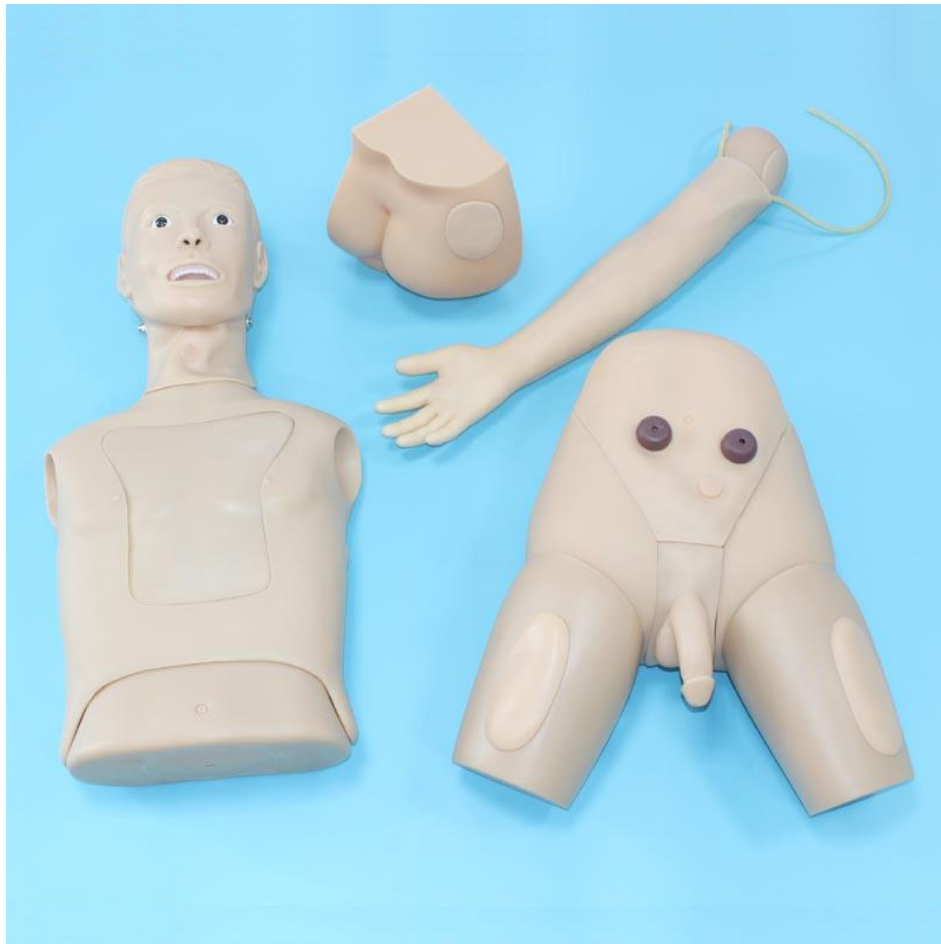

**Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.**

**Ada Med Supply Limited**

## **BIX - H70 ADVANCED BASIC NURSING TRAINING MODEL**

### **Product Features**

Based nursing operation, and clinical practice nursing technology outline requirements, this product is newly developed. It is an advanced type of basic nursing manikin, which cover all the nursing operation function model, instead of the before one, which can't really operate training nursing function.

This model is made up of plastic material, mold casting molding. Also, the injection, piercing, genital parts are made from imported soft plastic, etc. Image is clear, with real operation, easy disassembling, have reasonable structure, and is durable, etc. This model has all the functions, and some parts could be disassembled for general nursing teaching training. This is a currently domestic advanced nursing training model, with complete function, material.

### **Functions**

1. Hair and face washing
2. Eye and ear washing and administering
3. Oral care
4. Oxygen inhaling
5. Endotracheal intubation
6. nasal feeding
7. Gastrolavage
8. Endotracheal intubation
9. venipuncture, injection, blood transfusion (Arm)
10. Deltoid subcutaneous injection
11. hypodermic injection
12. intravenous injection and infusion
13. Intravenous infusion&Intravenous blood transfusion
14. Coloclysis
15. Intravenous puncture
14. Male/female urethral catheterization

15.male/ female bladder irrigation

16.Ostomy

17.Buttocks intramuscular injection

### **Application range**

1. Medical university, college of nursing, health school, the teaching of clinical medical colleges and universities teaching and student practical training.
2. For the hospital medical and nursing staff to continue education in clinical teaching practice training.
3. For popular clinical medical health units at the grass-roots level training.

### **Major Items**

1. During catheter operation, please smear lubricant on the catheter (such as detergent, soap and water, etc.).
- 2.For nasal feeding operation as well as gastric lavage, enema, male and female urethral catheterization. after the operation training, please empty the stomach, intestines, bladder residual liquid.
3. If no use the model for long time, the model should be wiped clean, with large packaging store in a cool and dry place, to extend the service life.

**Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.**

**Ada Med Supply Limited**

---

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd. Building 3,  
No.1288 kungang Road, Xiaokunshan Town, Songjiang District, Shanghai, China

Tel: +86-13383897707

Fax: 021—57722933

Post code: 201612

E-mail: [edith@adahealthy.com](mailto:edith@adahealthy.com)

Website: [www.adahealthy.com](http://www.adahealthy.com)