


# MANUAL

## **BIX/F6 Delivery mechanism teaching models Instructions**

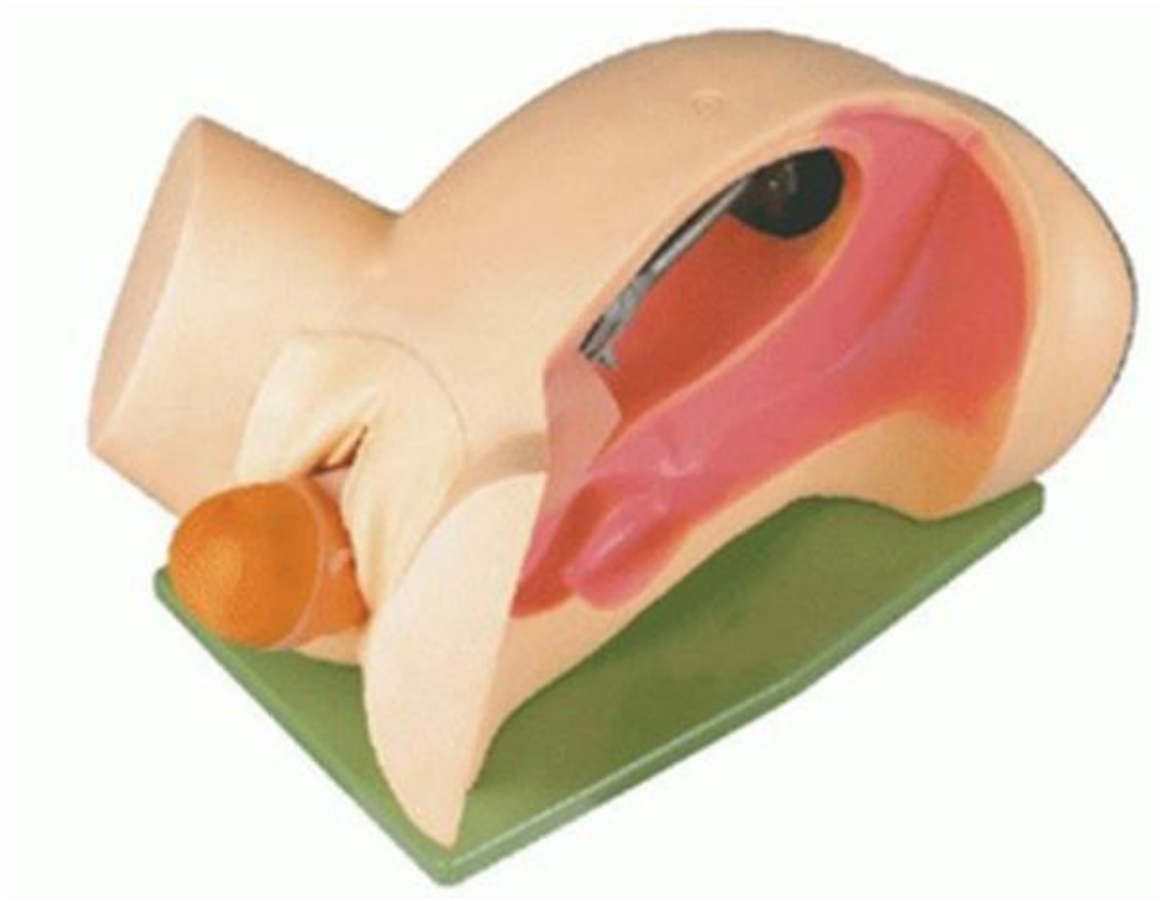

**Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.**

**Ada Med Supply Limited**

## BIX/F6 Delivery mechanism teaching models Instructions

### 一、Model Uses

This model is applicable to medical colleges and obstetrics and gynecology, midwifery, health, school nurses obstetrics and gynecology as a delivery mechanism to explain the normal visual and presentation aids rpm. And to protect the perineum as the baby is delivered by.

### 二、Teach Content:

Model machine into normal delivery basis, to demonstrate delivery of Cohesion - fall - flexion - rotating - Yang Shen - reset and external rotation - shoulder turn delivered and other normal machine operation.

### 三、Instructions:

When placing a metal plate inside the handle, into the rocking shaft and then slowly clockwise cranking, when the left front position of the fetal head delivery of the vulva, fetal face and then turned to the left a schematic of the completed delivery mechanism The whole process, stop cranking. (When the delivery of the fetal head can no longer shake out, in order to avoid mechanical damage.)

After teaching is completed, the slow cranking the handle counterclockwise, when the fetus whole body turned to the right place to stop shaking. (Note: If you feel resistance when the fetal head is retracted vulva, can head to the left to gently care delivery.)

- 1.Except for the motor part made of metal, the rest are made of foam rubber and transparent plastic
- 2.When the metal structure to use, not to force shake turn, then slowly carried out, such as when generating resistance is not flexible, can view the structure whether or Zhazhu off, as long as you can dial positive

#### **四、Size:**

Nature Size

#### **五、Maintenance Warranty**

- 1.after the end of the model operation training, store in a cool dry place, avoid reducing the service life
- 2.The warranty period of one year, and lifetime maintenance

**Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.**

**Ada Med Supply Limited**

---

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd. Building 3,  
No.1288 kungang Road, Xiaokunshan Town, Songjiang District, Shanghai, China

Tel: +86-13383897707

Fax: 021—57722933

Post code: 201612

E-mail: [edith@adahealthy.com](mailto:edith@adahealthy.com)

Website: [www.adahealthy.com](http://www.adahealthy.com)