

MANUAL

BIX-H5T BUTTOCK INTRAMUSCULAR INJECTION AND COMPARED SIMULATOR



Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.

Ada Med Supply Limited

BIX-H5T BUTTOCK INTRAMUSCULAR INJECTION AND COMPARED SIMULATOR

Instructions:

A.Function and feature

- 1.Emulational adult hip, life-like model, excellent quality, refined and smooth skin texture
- 2.Distinct character: proximal femur, femoral great trochanter , posterior superior iliac spine, sacrum and tailbone
- 3.Excellent quality of emulational, high elasticity, real touch
- 4.Needle-proof material, high elastic skin ,long service time
- 5.Ability to intramuscular inject in three ways: gluteus maximus, gluteus medius, gluteus minimus

B.Application method

- 1.To take out the model of intramuscular injection from the suitcase;
- 2.To prepare the syringes and No.6-7 syringe needle
- 3.To check gluteus maximus, the rules of intramuscular injection include:
 - ①The UOQB method: drawing the horizontal line from the top of anal cleft to the left(or right), drawing the vertical line from the top of the crista iliaca, 1/4 upper area regards as the injection area.
 - ②The UOAL method: to separate the area between the anterior superior spine and coccyx into three equal parts, the top part regards as the injection area.
- 4.Using the right hand to hold the syringe as grip a pen, to use the middle finger to stabilize the syringe pintle, use the left hand to stabilize the skin, put the needle into the muscle vertically, make sure 2/3 of the length of needle into skin.
- 5.When the light goes red, it means stick too deep or wrong area; when the light goes right, it means operation is correct.

C.Maintain

- 1.To use the neutral detergent when the model is dirty (abluent, washing powder)

- 2.Keep the sharp items away to avoid model broken
- 3.Please store the model in cool and dry place if you do not use it.

Chinon Medical Model & Equipment Manufacturing Co.,Ltd.
Ada Med Supply Limited

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd. Building 3,
No.1288 kungang Road, Xiaokunshan Town, Songjiang District, Shanghai, China

Tel: +86-13383897707

Fax: 021—57722933

Post code: 201612

E-mail: edith@adahealthy.com

Website: www.adahealthy.com