

MANUAL

BIX-J3A Infant Airway Management Model



Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.

Ada Med Supply Limited

BIX-J3A Infant Airway Management Model

About The endotracheal tube whether Inserted into the trachea or not , there are three check methods as follows:

1. Visible the endotracheal catheter model in the tracheal bronchia through in the cross profile of the head & neck.
2. Stand the head model , pull open the wall of the throat through the hole in the back of the head to check it.
3. Use the mask of the Balloon mask resuscitator Connect trachea catheter, to pressurize, lung model will be enlarged, prove the catheter was inserted into the trachea.

Features:

1. Realistic neonatal anatomical structure;
2. Intubation through the mouth & nose trachea;
3. The simulation of material and function;
4. You can test whether the intubation is inserted correctly by blowing airway;
5. Simulate the expansion of the lung;

Operation mode

Spray lubricant in one side of the nostrils at first when intubation through the nose , Then insert the endotracheal tube to the side nostrils, and down through nasal passages into the throat, use intubating forceps lead the endotracheal tube into trachea under laryngoscope look, The specific method see details “ Neonatal asphyxia recovery set books ” relevant chapters. The check method is same with oral intubation.

Cleaning and maintenance:

Take down the head model from the scaffold after using, rinse it after scrubbing with warm soapy water or dilute cleaning solution. Some pollution can be erased immediately use ethanol, When necessary, the head inside can be scrubbing or washed use disinfectant, After natural drying to installed on the bracket afresh, lid and store it in a cool and dry place.

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.

Ada Med Supply Limited

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd. Building 3,
No.1288 kungang Road, Xiaokunshan Town, Songjiang District, Shanghai, China

Tel: +86-13383897707

Fax: 021—57722933

Post code: 201612

E-mail: edith@adahealthy.com

Website: www.adahealthy.com