

# MANUAL

## **BIX-H4T Advanced Bottom Injection And Anatomy Model**



**Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.**

**Ada Med Supply Limited**

## BIX-H4T Advanced Bottom Injection And Anatomy Model

### 一、 Model Features:

This is our latest development model for human bottom injection.

The image is lifelike、 materials are durable.Made of silicon material,mold by casting process.

This model is an ideal medical product to do clinical skill training used for National Medical Colleges and University、 College of Nursing、 Health School and Clinical Medical Staff.

### 二、 Main Function Characteristics:

Anatomy structure is accurate and clear,to provide a scientific basis for more accurate injection operation.

Structure comprising: a proximal femur, trochanter, the anterior superior iliac spine, posterior superior iliac spine and sacrum.

Outer left hip on a quarter removable, easy to observe their internal structure confirmation, gluteus medius, gluteus maximus muscles, sciatic nerve and vascular structures.

Available for 3 methods of injection training: backside buttocks injection method, injection method and the lateral ventral hip bone muscle injection.

This model use imported plastic elastomer material, made of stainless steel mold pouring from high temperature, feel the texture of real, durable, needle marks are not clear visible.

### 三、 Using Method:

1. The most common injection position is the gluteus maximus, also can choose the gluteus medius, gluteus minimus muscle, the vastus lateralis muscle and the deltoid, etc.

The division method of gluteus maximus injection is: ① Cross Method: draw a horizontal line from the apex of the gluteal cleft in left (or right), in which the iliac crest highest stipple a perpendicular bisector, outside the top 1/4 of the injection zone  
② Connection method: the anterior superior iliac spine and tail sacral divided into three equal parts at the wiring, external injection zone on 1/3.

### 三、 Maintenance、 Warranty

- 1、 Keep the model clean, after the operating, model should be wiped clean.
- 2、 If not used for a long time, wipe clean the model, pack in carton and store in a ventilated, cool and dry place, in order to prolong the service life.
- 3、 This model on the purchase date for a year or the warranty series in the contract, if there is any quality problems caused by the manufacturing or the original material(except for man-made), we responsible for free maintenance service. We also promised to lifelong maintenance, charged by users.

**Chinon Medical Model & Equipment Manufacturing Co.,Ltd.**

**Ada Med Supply Limited**

---

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd. Building 3,  
No.1288 kungang Road, Xiaokunshan Town, Songjiang District, Shanghai, China

Tel: +86-13383897707

Fax: 021—57722933

Post code: 201612

E-mail: edith@adahealthy.com

Website: www.adahealthy.com