

MANUAL

BIX/CPR100A Half body CPR training manikin(Simple electronic)



Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.


Ada Med Supply Limited

BIX/CPR100A Half body CPR training manikin(Simple electronic)

CONTENTS

Check out the contents of the box. It should contain: one manikin torso, one manikin head, 10 face shield lung bags.

PARTS

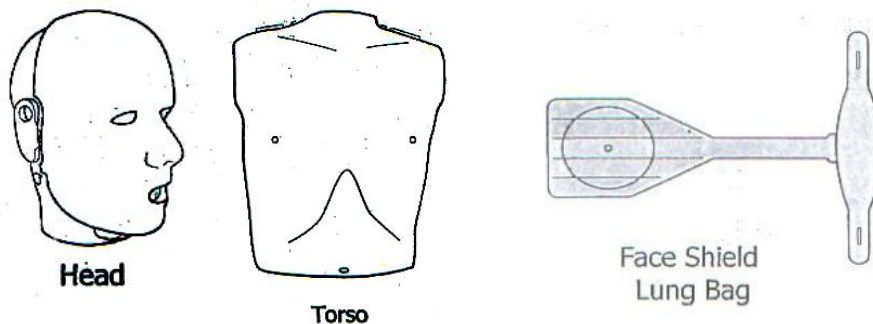

ASSEMBLY

- 1、 Open the torso of the manikin by pulling the "tabs" located on the shoulder and lift up the torso skin [PIC.1]
- 2、 Attach the head to the torso by placing the neck of the head onto the two spindles on the neck portion of the torso.[PIC.2]
- 3、 Fold the face shield lung bag(according to PIC.3)
- 4、 tilt the head of the manikin [PIC.4]
- 5、 Insert the lung bag,folded end,into the mouth of the manikin [PIC.5]
- 6、 Push the lung bag into the mouth until the bag exits through the neck [PIC.6]
- 7、 Grasp the lung bag and pull it through the neck. [PIC.7]
- 8、 The face shield lung bag is then placed on the top of the compression plate (the hard plastic piece found on top of the torso foam core). [PIC.8]

9、 Make certain that there are no twists in the lung bag.

10、 Attach the face shield tabs to the "buttons" located on the side of the manikin head. [PIC.9]

11、 The torso may then be closed by pushing down on the shoulder tabs until they snap into place.



Figure 1

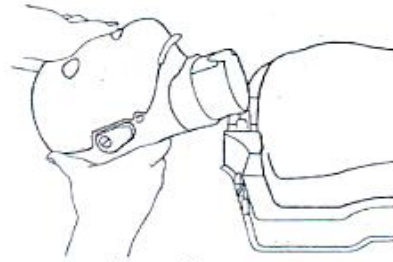


Figure 2

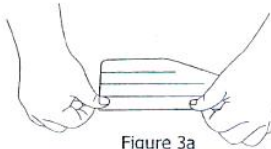


Figure 3a

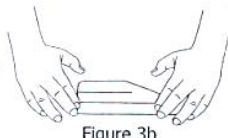


Figure 3b

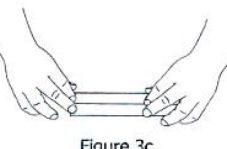


Figure 3c

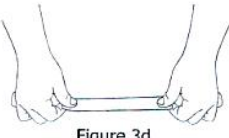


Figure 3d

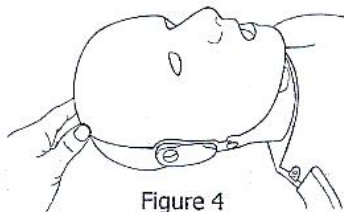


Figure 4

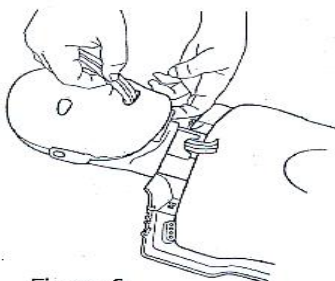


Figure 6

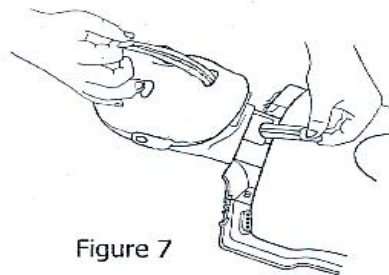


Figure 7

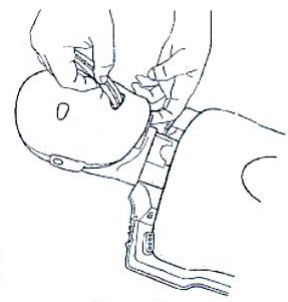


Figure 5

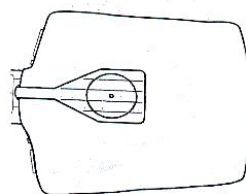



Figure 8



LED LIGHTS

- 1、 All the 3 lights will be lighted when your first correct compression. That means the battery is fully charged.
- 2、 If not, please check the depth of your compression. Try harder until you can hear the click.
- 3、 In case your compression is right, make sure if the battery is well put.
- 4、 Once you start your compressing, the yellow and the red LED Light will be turned off.
- 5、 If the green LED Light is not lighted, please try harder until you can hear the click.
- 6、 When the operating is done over 10 seconds, the yellow light will be flashing, then the red light will be lighted, soon it goes to the hibernation.

BIX/CPR 100A CPR Training Manikin

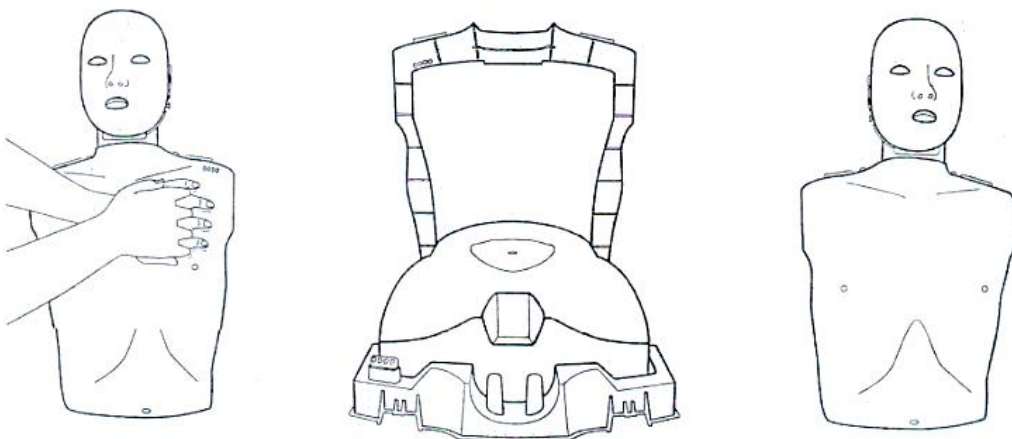

START HERE

- Using this instruction sheet as your first source of information about the BIX/CPR 100 CPR Manikin.
- Clean the manikin face skin and the manikin torso skin prior to using in an actual class

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.

Ada Med Supply Limited

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd. Building 3,
No.1288 kungang Road, Xiaokunshan Town, Songjiang District, Shanghai, China

Tel: +86-13383897707

Fax: 021—57722933

Post code: 201612

E-mail: edith@adahealthy.com

Website: www.adahealthy.com