

MANUAL

BIX/F9B Advanced Female Contraceptive Practice Model



Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.

Ada Med Supply Limited

BIX/F9B Advanced Female Contraceptive Practice Model

1. Product features

The BIX-F9B female intrauterine device and training model are family planning teaching aids designed and developed to guide and train the placement and removal of intrauterine devices and meet the needs of medical teaching and training. This model is based on the anatomy of normal human female reproductive organs and can display anatomical structures such as the vagina and uterus. Its characteristic is that the uterus is in a coronal section and is covered by a transparent plastic window. The entire process of placement and removal of the intrauterine device can be observed under direct vision. The vagina can be dilated, and a vaginal speculum can be used to simulate and demonstrate the insertion and removal techniques of a female intrauterine device. The model has a realistic image and a real feeling. It is suitable for clinical teaching in medical schools, nursing schools, medical technical schools, health schools and other professional clinical teaching and practical training of family planning for students. It is also an ideal internship model for family planning teaching and training of clinical medical staff.

2. Usage method

The characteristic of this product is that the front part of the uterus is transparent, and the entire process of placement and removal of the intrauterine device can be observed under direct vision. Use one hand to insert a vaginal speculum into the vagina to expand the vagina, and use the other hand to hold the contraceptive device inserter and special intrauterine contraceptive device or ring, insert it into the uterus from the uterine opening, and perform technical operation training on placement and removal. It can also operate and train contraceptive tools such as female condoms, contraceptive sponge plugs and vaginal diaphragms (diaphragms).

3. Maintenance and warranty

- (1) After the model training operation is completed, it can be washed with soap and clean water and dried.
- (2) When not in use for a long time, the model should be wiped clean, packed and stored in a cool and dry place to extend its service life.
- (3) This model is warranted with the purchase invoice and warranty card. The free warranty period is one year and lifetime maintenance.

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.

Ada Med Supply Limited

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd. Building 3,
No.1288 kungang Road, Xiaokunshan Town, Songjiang District, Shanghai, China

Tel: +86-13383897707

Fax: 021—57722933

Post code: 201612

E-mail: edith@adahealthy.com

Website: www.adahealthy.com