


MANUAL

BIX/FS Advanced gynecological examination model

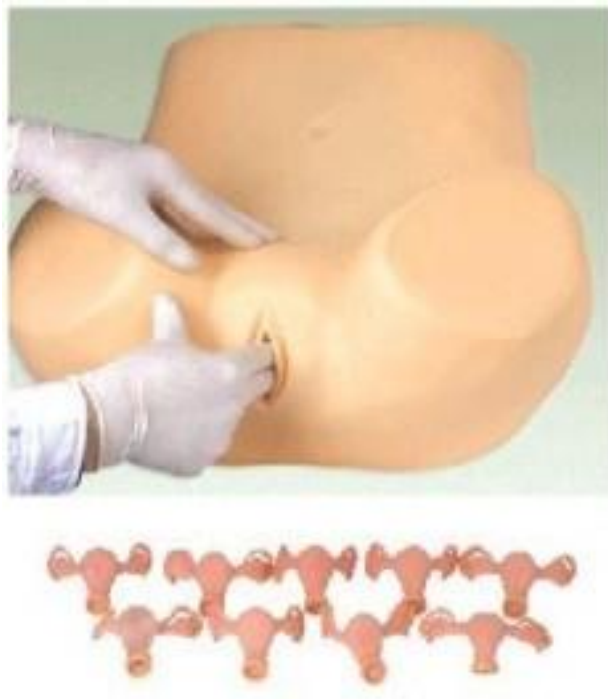

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.

Ada Med Supply Limited

BIX/FS Advanced gynecological examination model

A. Model Uses

Advanced gynecological examination model according to teaching requirements, engineered simulation results, training for teachers and students to demonstrate in the examination and speculum examination. It applies to medical institutions, MCH operating teach and practice.

B. Main Function

The model for the adult female torso from the lower half of the abdominal, pelvic two parts. It can be used for the following functions:

1. Normal and various abnormal uterine palpation.
2. Rectovaginal, triple diagnosis examination.
3. Vaginal speculum examination and colposcopy.
4. Naked eye normal and various abnormal cervical lesions.
5. IUD placement and removal.
6. Size and position of the diaphragm.
7. Observation of the uterus, ovaries, fallopian tubes, round ligaments and other anatomical structures in the pelvis.
8. Adjustable tilt angle position around the uterus.
9. Pelvic measurements.

C. Instructions

1. Vaginal examination: vaginal speculum is inserted, the expansion of the vagina, cervix exposed.
2. Doublet rash, rash triple, referral rectal examination: the length of the vagina can be displayed, cervix, mouth shape, uterine attachment.
3. 6cervical lesion: erosion, linear lacerations, acute purulent cervicitis, cancer, polyps, cervical gland infection (follicular) cyst, two normal uterus

D.Product Care

- 1.Keep the model clean after the operation training should model with water was clean cloth to scrub, scrub prohibited chemical solvents.
- 2.Unused for long periods,Please clean up the model, packing stored in a ventilated, cool dry place for long life.
3. In this model, the date of purchase within one year of the warranty period specified in the contract or, if due to any manufacturing process or originals (except human-caused) the failure of quality problems, the company responsible for the free repair service. And promised lifetime maintenance(User fees conceited).

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.

Ada Med Supply Limited

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd. Building 3,
No.1288 kungang Road, Xiaokunshan Town, Songjiang District, Shanghai, China

Tel: +86-13383897707

Fax: 021—57722933

Post code: 201612

E-mail: edith@adahealthy.com

Website: www.adahealthy.com