

# MANUAL

# **BIX-FS9 Advanced Infant Full-body Venipuncture Manikin**



Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.

Ada Med Supply Limited



## **BIX-FS9 Advanced Infant Full-body Venipuncture Manikin**

The model consists of a stainless steel mold, pouring made with imported high temperature plastic materials, Has applied nursing teaching material soft, feel real, real operating characteristics.

#### Main function:

- 1. Baby intravenous mainly head and neck vein venipuncture injection.
- 2. Cubital about arms, armpits, and the hand vein injection.
- 3.Intravenous injection of umbilical, blood, catheter insertion puncture exercises.
- 4.Right leg great saphenous vein, small saphenous vein injection, infusion (blood) puncture practice
- 5. Buttock Intramuscular injection practice.
- 6. Nose and open mouth can be a feeding tube, stomach tube intubation exercises, pumping exercises, arm while feeding tube can be fixed, dressing, cleaning, nursing and other operations.
- 7. Needle obvious sense of frustration, when the correct puncture, blood produce a significant return.
- 8. Veins and blood vessels and skin still the same skin puncture site can withstand repeated hundreds of times not leak puncture.

### Method of application

Take out the baby head injection module, Clamping the tube of one side, Injected the simulated blood into the other side of the tube(Blood powder and water to reconcile).

### **Cleaning and Maintenance**

After the model used, can be removed from the stand, scrub by warm soapy water or diluted cleaning solution rinse with water.

Some of the pollution can be cleaned with alcohol immediately, If necessary, it can also be used in the rinse disinfectant scrub head, after drying, re-mounted on the stand, capped stored in a cool dry place.



# Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd. Ada Med Supply Limited

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd. Building 3,

No.1288 kungang Road, Xiaokunshan Town, Songjiang District, Shanghai, China

Tel: +86-13383897707

Fax: 021—57722933

Post code: 201612

E-mail: edith@adahealthy.com

Website: www.adahealthy.com